Motorhandstück 38

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NOUVAG® SWISS
Dental and Medical Equipment QUALITY

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- Nouvag AG can not be held liable for any malfunction of the device performance failure, its designed utility and/ or when the device is miss-used or not operated in accordance with the user instructions set out in Operating Manual.
- When storing the instrument without a clamped tool for long periods of time, the chuck must be unclamped!
- Run hand piece only with clamped tool!

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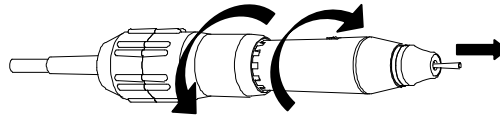
Do not remove the chuck while the Motor is running!

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The chuck 2.35 mm is usually included with the motor handpiece. The chuck can be opened and closed by twisting the hanpiece front part and back part:

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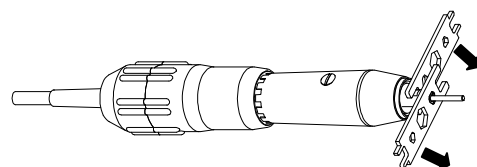
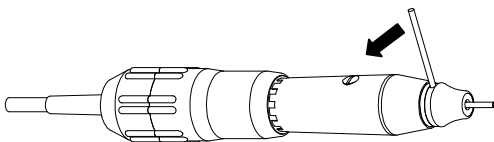
- ...clockwise to open chuck
→ grinder can be changed
- ...counterclockwise to close chuck
→ grinder fixed

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- 1) Closed chuck with inserted dowel pin \varnothing 2.35mm (included)
- 2) Insert the end of the pin \varnothing 2.5mm (included) into the hole and loosen tip by pressing counterclockwise.

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Without clamped dowel pin, the chuck can get damage and become deformed due to pressure!

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- 3) Loosen chuck with help of the 2 keys (included) → hold shaft firmly with one key and unscrew chuck with the other one.
- 4) Insert and tighten new chuck

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Assembling after the change is similar, but in reverse order and counterclockwise

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- Motorhandpiece 38 is not sterilizable!
- Do not clean Motorhandpiece 38 with compressed air!
- Do not bend the motor cable!

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The Motorhandpiece, cable and plug should be cleaned after each use to prevent the internal build-up of deposits and debris that can destroy it, if not removed. Use the brush and wipe it with a clean, damp cloth.

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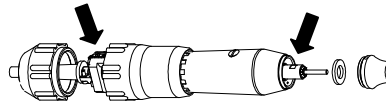
Remove weekly carbon and grinding dust with the brush!

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
Clean with the enclosed brush the soiled motor chamber. The same procedure applies for the chuck chamber. **The dusty felt packing in the chuck chamber must be cleaned and replaced when necessary.** The tip and motor cap can be screwed or unscrewed respectively

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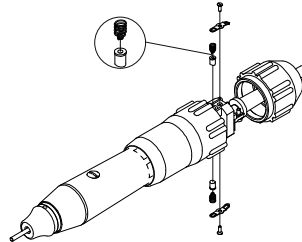Changing the carbon brushes:

- 1) Unscrew motor cover
- 2) Remove the screws on the side with the appropriate screwdriver
- 3) Remove wore out carbon brushes and springs
- 4) Insert new carbon brushes and springs
→ **curved carbon brushes end must rest on the collector**
- 5) Tighten screws on the side, again
- 6) Screw motor cover, again

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Problem:	Cause:	Solution:
Motor does not run	Plug is not inserted properly	Insert plug and check fitting
Motor stops when cable moves	Defective cable	Replace cable
Sporadic motor disfunction	Worn out brushes	Replace brushes

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Weight:..... 375 g
 Max. output: 80 W
 Max current: 4 A
 Rated voltage:30 V
 Rated speed:..... 35,000 rpm
 Cable length.....2 m
 Chuck diameter standard:..... 2,35 mm
 Chuck diameter optional..... 1,6 / 3 / 3,175 mm

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Art.-Nr	Description	Units
1115.....	Carbons brushes with spring	two pieces each
1198.....	Brush.....	1 piece
20593.....	Key for chuck.....	2 pieces
20572.....	Felt packing.....	1 piece
20165.....	Dowel pin Ø 2.5mm.....	1 piece

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www.nouvag.com

All NOUVAG-Service Centers worldwide pls. see at

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Disposal:

Disposal of device, components and accessories must strictly conform to local laws and regulations as set out by the relevant authorities.

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NOUVAG AG
NOUVAG USA Inc
NOUVAG GmbH