STAND

DESCRIPTION OF ASSEMBLY AND OPERATING INSTRUCTIONS

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The STAND is a mobile device carrier. It's equipped with a bottom shelf and a top shelf, on which different devices can be placed. The STAND comes with 4 antistatic rolls, from which 2 are lockable.

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- **Nouvag AG can not be held liable for any malfunction of this mobile device, performance failure, and/or its designed utility, nor can Nouvag AG be held liable for any injuries, in any case when the device is missused or not operated, applied or maintained in strict accordance with the user/owner instructions set out in the Operating instructions.**
- **Dismantle the STAND for transportation.**

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Stand foot.....	1 piece
Stand column.....	1 piece
Shelf.....	1 piece
Star handle with spacer.....	2 pieces
Antistatic rolls with integrated mounting screws.....	2 pieces
Antistatic rolls with integrated mounting screws and brakes.....	2 pieces
Allan screw M10 x 35 mm.....	4 pieces
Allan screw M4 x 16 mm.....	4 pieces
Washer M10.....	4 pieces
Washer M4.....	8 pieces
Allan wrench 8 mm.....	1 piece
Allan wrench 5 mm.....	1 piece
Allan wrench 3 mm.....	1 piece

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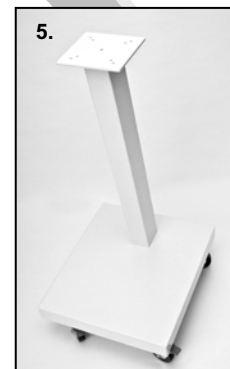
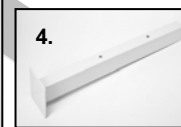
Only clean the surface of the unit, do not use harsh cleaners or solvents for cleaning. Use 80% ethyl alcohol or microbiologically effective disinfectants.

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1. View: Stand foot.
2. Connect rolls with integrated Allan screws with Stand foot (Allan wrench 5 mm). Mount rolls with brakes on the front side.
3. View: Stand foot with mounted rolls.
4. Connect column with stand foot with screws M10x35mm and washer (Allan wrench 8 mm).
5. View: Stand foot with rolls and fixed column.
6. Connect shelf with column with screws M4x16mm and washer (Allan wrench 3 mm).
7. View: Stand foot with rolls, fixed column and shelf.
8. Connect star handles with spacer with stand column.
9. View: Complete Stand.

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www.nouvag.com

All NOUVAG-Service Centers worldwide please see at

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Disposal:

Disposal of device, components and accessories must strictly conform to local laws and regulations as set out by the relevant authorities.

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NOUVAG AG • St. Gallerstrasse 23-25 • CH-9403 Goldach
NOUVAG USA, LLC
NOUVAG GmbH • Schulthaiß-Strasse 15 • D-78462 Konstanz